



Instructions for Embryo Flushing Plus

Embryo Flushing Plus is simple and easy to use!

Embryo Flushing Plus achieves embryo collection by flushing the uterus of a superovulated donor cow

1. Embryo Flushing Plus can be kept at room temperature, but it may also be refrigerated at 2 to 8°C.
2. Hang the bag on a hook at a certain height before embryo flushing.
3. Insert the needle into the needle septum port, or link the flushing tube with the spike port.
4. Remove air bubbles from the flushing tube.

Embryo flushing is now ready.

Use Embryo Flushing Supreme by its expiration date. Discard if any precipitation forms.

Please contact Renova Life, Inc. to discuss any problems by calling 301-405-0298 or sending an email to info@renovalife.com.