



## Instructions for Embryo Flushing Supreme

**Embryo Flushing Supreme is simple and easy to use!**

**Embryo Flushing Supreme achieves embryo collection by flushing the uterus of a superovulated donor cow**

1. Embryo Flushing Supreme may be refrigerated at 2 to 8°C.
2. Pre-warm Embryo Flushing Supreme at room temperature before embryo flushing.
3. Hang the bag on a hook at a certain height.
4. Insert the needle into the needle septum port, or link the flushing tube with the spike port.
5. Remove air bubbles from the flushing tube.

Embryo flushing is now ready.

Use Embryo Flushing Supreme by its expiration date. Discard if any precipitation forms.

Please contact Renova Life, Inc. to discuss any problems by calling 301-405-0298 or sending an email to [info@renovalife.com](mailto:info@renovalife.com).