

Instruction for Embryo Rinse Plus

Embryo Rinse Plus is simple and easy to use!

Rinse the filter and embryo after flushing the donor cows

1. Embryo Rinse Plus (ERP) can be kept at room temperature, but it may also be refrigerated at 2-8 °C.
2. For 500 mL ERP transfusion bag, after embryo flushing, hang the bag on the hook with a certain height. Insert the needle into needle septum port or link the rinsing tube with spike port. Remove the air bubbles from the rinse tube. Rinse the filter to remove blood clots. Bring the filter into lab for embryo search.
3. For 500 mL ERP bottle, in the embryo laboratory, open the cap of bottle, rinse the filter or embryos in the petri dishes. Transfer the embryos from one petri dish to another that contains new ERP medium. This can remove any small blood clots and unknown/unnecessary elements from the uterus.

Use Embryo Rinse Plus by its expiration date. Discard it if any precipitation forms or a dramatic color change is observed.

Please contact Renova Life Inc. to discuss any problems by calling 301-405-0298 or sending an email to info@renovalife.com.