



Instructions for Oocyte Aspiration Plus

Oocyte Aspiration Plus is simple and easy to use!

Oocyte Aspiration Plus achieves ovum aspiration by ovum pickup (OPU) from the ovarian follicles.

1. Oocyte Aspiration Plus can be refrigerated at 2 to 8°C.
2. Pre-warm Oocyte Aspiration Plus at room temperature before performing OPU.
3. Hang the bag on a hook at a certain height.
4. Insert the needle into a needle septum port or link the tubing with a spike port.
5. Remove air bubbles from the tubing.
6. OPU can now be performed.

Use Oocyte Aspiration Plus by its expiration date. Discard if any precipitation forms.

Please contact Renova Life, Inc. to discuss any problems by calling 301-405-0298 or sending an email to info@renovalife.com.